



CAN-DAN REHATEC LTD.
3-1378 Sandhill Drive,
Ancaster, Ontario L9G 4V5

Phone: (905) 648-7522
Fax: (905) 648-7799

Manufactured by

CAN-DAN REHATEC LTD.
3-1378 Sandhill Drive,
Ancaster, Ontario L9G 4V5
Phone: (905) 648-7522
Fax: (905) 648-7799

Dealer

INSTRUCTION GUIDE

**READ THOROUGHLY BEFORE OPERATING
MOBILE ELECTRIC HEIGHT ADJUSTABLE CHANGE TABLE.**

Model # 19-97-62
Size: 62" X 28"

Model # 19-97-72
Size: 72" X 28"

Model #20-02-62
Size: 62" X 30"

Model #20-02-72
Size: 72" X 30"

Model #20-03-62
Size: 62" X 32"

Model #20-03-72
Size: 72" x 32"

Manufactured in Canada by Can-Dan Rehatec Ltd.

Instruction Guide

Table of contents.

1. Safety Precautions.
2. Scope of Application.
3. Product Information
4. Operation and Handling Instruction
5. Maintenance
6. Trouble Shooting
7. Warranty.

Scope of Application

Safety Precautions

Before using, always make certain:

1. Person (care giver) have received appropriate instruction and training.
2. The Instruction Guide has been carefully studied and understood.
3. The wheels are in the locked position.
4. The user is secured to the table with the straps.
5. At anytime - **DO NOT** store anything under the table-top.

NEVER leave user unattended on the table.

- ✓ When raising the Mobile Table, be certain to keep each end of the table sufficiently away from side walls, as table raises at a slight left to right angle. (Total 12 ")
- ✓ Require a GFI receptacle (15 amp).


Maximum Lifting Capacity: 250 lbs

The Mobile Electric Height Adjustable Changing Table is a lifting device. Through the use of an electrically powered hand-control which is connected to a 24 V actuator, they are lowered to the height of the wheelchair. The care-giver can then slide the individual out of the wheelchair and onto the table keeping back strain to an absolute minimum.

Once on the table, the individual can be secured with the two (2) integrated safety belts.

With a lifting capacity of 250 lbs, it is suitable for use in schools, group homes, private homes, clinics and nursing homes and other care-giving situations.

Product Information

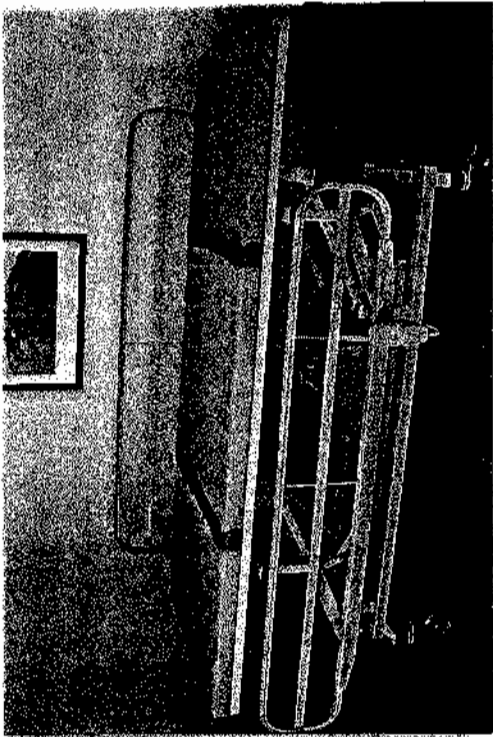

Above picture shows Change Table with optional "Side-Rails"

Model #	Size	Model #	Size	Model #	Size
19-97-62	62" x 28"	20-02-62	62" x 30"	20-03-62	62" x 32"
19-97-72	72" x 28"	20-02-72	72" x 30"	20-03-72	72" x 32"

Specific Feature

- Adjustable in height from 22 inches to 37 inches
- The padded top comes complete with two (2) body safety belts.
- Electric hand control fastens to a special hanger on front of the table when not in use.
- Casters with brakes for safety.
- Powered by a 115VAC to 24VDC actuator
- Mattress is germ resistant, fire retardant and made of vinyl for easy cleaning.

BE GOOD TO YOUR BACK - WHEN HELPING OTHERS.

Manufactured in Canada by Can-Dan Rehabec Ltd.

Operation and Handling Instructions

Simply lower the table, transfer the person onto the padded table top, secure the two body safety belts and raise the table to your most comfortable working position.

Before using always make certain:

- ✓ Persons have received appropriate instruction and training.
- ✓ The instruction Guide has been carefully studied and understood.
- ✓ On Mobile Change Table make certain the wheels are in the locked position.
- ✓ At any time do not store anything under the table top.

NEVER leave user unattended on table.

When raising the Mobile Change Table, be certain to keep each end of the table sufficiently away from side walls, as table raises at a slight left to right angle(12")

GFI outlets are necessary when used in a wet environment or when required by electrical code.

Maintenance for Mobile Change Table.

Check following regularly:

Check safety belts for signs of wear.

Check wheels and brakes for proper functioning.
Clean as needed to ensure proper functioning.

Check list for yearly maintenance:

- Safety Belts
- Wheels incl. 2 way brakes.
- Clean velcro on table top and mattress.
- All moving parts a few drops of lubricating oil
(Do not use WD40)

Date: _____

Trouble - Shooting

If the table fails to operate ...

Check the following:

- * The table is plugged into the AC outlet.
- * There is power to the AC outlet.
- * Green light on power supply (indicating power from your AC outlet)

If the handcontrol fails to operate

Check the following:

- * Make certain that handcontrol is plugged into the power supply box.

For other service related problems, contact your local Can-Den Rehatec representative.

Description of WARRANTY.

Can-Dan Rehatec Ltd. as manufacturer, warrants from the **DATE OF DELIVERY TO THE FIRST CONSUMER** that each item described below, as **NEW AND UNUSED**, will be free from any defects in material and/or workmanship for a period of:

- Two (2) year on Change Table Frame.
(Manufactured by Can-Dan Rehatec Ltd.)
- 1 (one) year on Linek Actuator
- Six (6) consecutive months for accessories.

Accessories includes: Hand Control.
Table Top.
Mattress.
Safety Belts.
Wheels.

Defective parts must be returned to: **Can-Dan Rehatec Ltd.**
All parts replaced under this warranty become the property of:

Can-Dan Rehatec Ltd. Ancaster, Ontario, Canada

The warranty specifically excludes:

- Normal wear and tear items.
- Labor, parts and lubricant costs of all maintenance services.
- Installation done by a non-authorized representative.
- Damage resulting from improper repairs, modification or use of non-approved parts or, repairs done by a non-authorized **Can-Dan Rehatec** representative.
- Damage resulting from accident, fire, theft, vandalism or any Act of God.

Power Supply No. _____

Serial No. _____

Signature _____ Date _____

Change-Table Fact Sheet:

Mobile Model	Length: 62"	Length: 72"
Matt Size: Width	28" / 30" / 32"	28" / 30" / 32"
Height Adjustable Range	22" - 37"	22" - 37"
Space Requirement Length	74"	84"
Space Requirement Width	28" / 30" / 32"	28" / 30" / 32"

NOTE: All tables require an GFI Receptacle (15 amp)

For the Mobile Change Table the outlet should be no higher than 16" from the floor to the top of the outlet and preferably centered to the position of the table.