



**CAN-DAN REHATEC LTD.**

3-1378 Sandhill Drive,  
Ancaster, Ontario. L9G 4V5

Phone: (905) 648-7522  
Fax: (905) 648-7799

## **Wall Mount Electric Height Adjustable Changing Table**

**Model # 19-88-62**

**Size: 62" x 24" x 2"**

**Model # 19-88-72**

**Size: 72" x 24" x 2"**

### **Instruction Guide**

**Read Thoroughly Before Operating.**

**Wall Mount Electric Height Adjustable Changing Table  
Instruction Guide.**

**TABLE OF CONTENTS.**

- 1. Safety Precautions.**
- 2. Scope of Application.**
- 3. Product Information / Wall Mount Table**
- 4. Installation Instructions.**
- 5. Operation and Handling Instructions**
- 6. Maintenance.**
- 7. Warranty.**

## 2. Scope of Application.

The wall mount Changing Table is a lifting device. Through the use of an electrically powered hand control which is connected to a 24V actuator, they are lowered to the height of the wheelchair. The caregiver can then slide the individual out of the wheelchair and onto the table keeping back strain to an absolute minimum.

Once on the table, the individual can be secured with the two (2) integrated safety belts.

The caregiver can then raise the table to their ideal working height, eliminating back strain from bending during the changing and cleaning of the individual.

The wall-mounted version of the table comes equipped with wire storage baskets which can hold towels, diapers, talc and other necessary accessories for changing and cleaning.

With a lifting capacity of 250 lbs, it is suitable for use in private homes, clinics, nursing homes, schools and other caregiving situations.

## 1. Safety Precautions.

Before using, always make certain:

1. Persons have received appropriate instruction and training.
2. The Instruction Guide has been carefully studied and understood.
3. The user is secured to the table with the straps.


Never leave user unattended on table.

GFI outlets are necessary when using in a wet environment or when required by electrical code.

**Maximum Lifting Capacity: 250 lbs.**

### 3 Product Information

Simply lower the table, transfer the person onto the padded table top, secure the two body safety belts(if necessary) and raise the table to your most comfortable working position.

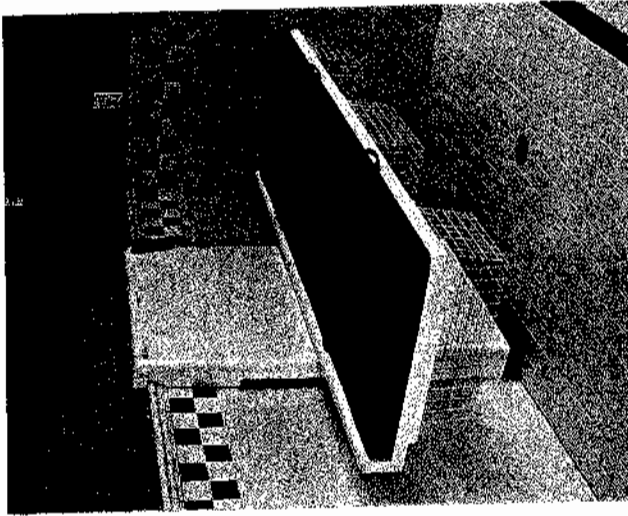

#### Specific features

- The padded top comes complete with (2) body safety belts.
- Three(3) wire storage baskets located under the table top are standard.
- Adjustable in height from 21 inches (floor to padded top) to 37 inches.
- Electric hand control can be fastened to front cover with velcro when not in use.
- Front cover is acrylic for easy cleaning.
- 110V/60HRZ operation.

3.

### 4. Installation Instruction

#### 1) For Block-Wall.

- ✓ Place MAIN frame against the wall.
- ✓ Adjust the legs to get frame in level (use a level)
- ✓ Hold frame against the wall and predrill with 3/8" drill - 4 holes in wall (to match holes already drilled on frame)
- ✓ Move the frame, drill with 5/8" drill and install "Rawl Anchor"
- ✓ Bring the frame back and fasten with 3/8" bolts.

#### 2) Power Supply.

Place front cover close up to frame and connect plug from front cover to the power supply box.

**NOTE:** In slot on box.

#### 3) Front Cover.

Place front cover over MAIN frame, predrill hole in top for 1/4" bolt, fasten with 1/4" top nut.

4.

6.

## Maintenance for Wall Mounted Change Table.

### Check following regularly:

Check safety belts for signs of wear.  
Clean velcro on table top and mattress

### To remove the front acrylic cover

Note that the hand control cable is mounted through the front cover and fastened to main frame.

**DO NOT REMOVE THE HAND CONTROL CABLE FROM THE FRONT COVER.**

Instead, unplug the hand control from the power supply box after lifting the front cover.

### Replacement of Hand Control

Cut out caulking and slide hand control cable out and away from cover.  
After installing new hand control, re-caulk.

### Lubrication of Wall Mounted Frame.

One (1) time each year, REMOVE the Front Cover (following instructions above) and grease the steel track. **PLEASE** use only "White Lithium Grease."

### Check list for yearly maintenance:

- Safety Belts
- Clean velcro on table top and mattress.
- Lubricate track (follow instructions above)

6.

## 5. Operation and Handling Instructions

Simply lower the table, transfer the person onto the padded table top, secure the two body safety belts and raise the table to your most comfortable working position.

### Before using always make certain:

- ✓ Persons have received appropriate instruction and training.
- ✓ The Instruction Guide has been carefully studied and understood.
- ✓ The user is secured to the table with the straps.

### Never Leave user unattended on table.

GFI outlets are necessary when using in a wet environment or when required by electrical code.

5.

## Description of WARRANTY.

**Can-Dan Rehatec Ltd.** as manufacturer, warrants from the DATE OF DELIVERY TO THE FIRST CONSUMER that each item described below, as NEW AND UNUSED, will be free from any defects in material and/or workmanship for a period of:

Twelve (12) consecutive months for Change Table.

(Manufactured by Can-Dan Rehatec Ltd.)

Six (6) consecutive months for accessories.

Accessories includes: Table-Top.  
Mattress.  
Safety Belts.

Defective parts must be returned to: **Can-Dan Rehatec Ltd.**  
All parts replaced under this warranty become the property of:

**Can-Dan Rehatec Ltd.** Ancaster, Ontario, Canada.

The warranty specifically excludes:

Normal wear and tear items.

Labor, parts and lubricant costs of all maintenance services.

Installation done by a non-authorized representative.

Damage resulting from improper repairs, modification or use of non-approved parts or, repairs done by a non-authorized **Can-Dan Rehatec** representative.

Damage resulting from accident, fire, theft, vandalism or any Act of God.

Power Supply No:

Serial No:

Signature

Date