



MOBILE ELECTRIC HEIGHT ADJUSTABLE CHANGE TABLE

Model #19-97-62 Size: 62" x 28"

Model #19-97-72 Size: 72" x 28"

Model #20-02-62 Size: 62" x 30"

Model #20-02-72 Size: 72" x 30"

Model #20-03-62 Size: 62" x 32"

Model #20-03-72 Size: 72" x 32"

Model #20-20-36 Size 72" x 36"

Model #20-20-40 Size 72" x 40"

INSTRUCTION GUIDE

V 1.7

Read Thoroughly Before Operating

Manufactured in Canada by Can-Dan Rehatec Ltd.





TABLE OF CONTENTS

Safety Precautions

Scope of Application

Product Information

Caregiver/ User Operating and Handling Instructions

Maintenance

Troubleshooting

Warranty



SAFETY PRECAUTIONS

Please read these instructions carefully, note the safety instructions and warnings. Use the product correctly and with care for the purpose for which it is intended. Failure to do so may cause damage to property and/or personal injury. Please keep this instruction manual safe for future use.

WARNING: The warnings, cautions and instructions discussed in this instruction manual cannot cover all possible conditions and situations that may occur. It must be understood by the operator that common sense and caution are factors which cannot be built into this product, but must be supplied by the operator.

Before using, always make certain:

- Caregiver has received appropriate instructions and training
- The Instruction Guide has been carefully studied and understood

SAFETY INSTRUCTIONS AND WARNINGS:

- Use a qualified person to maintain the table in good condition. Keep it clean and free of debris for best and safest performance.
- Caregiver should never leave user unattended.
- Use this table for lifting purposes only. Do not use it for any other purposes it is not designed to perform.
- Do not make any modifications to the table.
- Ensure the wheels are in a locked position.
- Do not, at any time, leave anything under the change table.
- When raising the table, be certain to keep each end of the table sufficiently away from side walls, as the table moves right to left by 8" as it rises and lowers (allow for total clearance of 12" to avoid pinch points).
- Maximum Lifting Capacity: 450lbs.
- Requires a GFI receptacle (15 amp).



SAFETY BELTS - USAGE GUIDELINES

The safety belts are designed to assist in positioning the user during use of the change table when a caregiver is present.

****Important****

- Safety belts should only be used at the recommendation of a Registered Healthcare Provider (when available).
- These belts are not intended to be used as restraints.
- Always ensure that an attendant is present while the user is secured with the safety belts.
- Never leave the user unattended while the belts are in use.
- For the safety and well-being of the user, please adhere to these guidelines.

SCOPE OF APPLICATION

The table is designed for changing and lifting. Using the controller, the table can be raised or lowered to the user or caregiver's safe transfer height.

Unassisted Transfers: The user can independently lower the table to their safe transfer height using the controller.

Assisted Transfers: The height adjustability enables caregivers to assist with transfers safely, reducing the risk of strain or injury. This promotes ergonomic practices during transfers.

Once the individual is on the table, they can be safely and comfortably positioned with the option of utilizing the safety belts. The caregiver can raise the table to their preferred working height for safe patient handling.

With a lifting capacity of 450 lbs., it is suitable for use in public facilities, schools, clinics, private homes, group homes, care homes, and other caregiving situations.

PRODUCT INFORMATION



Model #	Size (L x W)	Model #	Size (L x W)	Model #	Size (L x W)	Model #	Size (L x W)
19-97-62	62" x 28"	20-02-62	62" x 30"	20-03-62	62" x 32"	20-20-36	72" x 36"
19-97-72	72" x 28"	20-02-72	72" x 30"	20-03-72	72" x 32"	20-20-40	72" x 40"

SPECIFIC FEATURES

- The padded top comes complete with two (2) integrated safety belts.
- Adjustable in height from 20" (floor to padded top) to 36".
- Powered by 110VAC to 24VDC power supply.
- Hand control mounts to the front of the table when not in use.
- Casters with brakes for safety.
- Mattress is germ resistant, fire retardant, and made of vinyl for easy cleaning.



CAREGIVER / USER OPERATING AND HANDLING INSTRUCTIONS

- Using the controller, the caregiver/user lowers the table to a height where they can easily and safely transfer themselves or the individual to the padded tabletop.
- Upon completion of the transfer, the caregiver has the option to safely position the individual using the two (2) integrated safety belts.
- Once the individual is safely transferred to the change table, the caregiver can proceed to raise the table to a comfortable working position.

before using, always make certain:

- The instruction guide has been carefully studied and understood.
- The caregiver/user has received the appropriate instruction and training.
- Make certain the wheels on the table are in a locked position.
- When raising the table, be certain to keep each end of the table sufficiently away from side walls, as the table moves right to left by 8" as it rises and lowers (allow for a total clearance of 12" to avoid pinch points).
- Ensure nothing is under the table.
- During an assisted transfer, never leave the user unattended.



MAINTENANCE

Check the following regularly:

- Check safety belts for signs of wear.
- Check wheels and brakes for proper functioning.
- Clean Velcro on tabletop and mattress.
- Check all cables for signs of wear.
- Ensure all moving parts are lubricated. (Do not use a penetrating oil i.e. WD40. A silicone lubricant is recommended).

Annual inspections are recommended. Contact your local service provider.

TROUBLESHOOTING

If the table fails to operate, check the following:

- The table is plugged into the power outlet properly.
- There is power to the power outlet.
- The green light located on the power supply is lit (indicating power from your power outlet).

If the hand control fails to operate, check the following:

- Make certain that the hand control is plugged into the power supply box properly.

For other service-related problems, contact your local Can-Dan Rehatec Ltd. Representative.



WARRANTY

Can-Dan Rehattec Ltd. as manufacturer warrants from the DATE OF DELIVERY TO THE FIRST CONSUMER that each item described below, as NEW AND UNUSED, will be free from any defects in material and/or workmanship for a period of:

- Four (4) years on change table frame (Manufactured by Can-Dan Rehattec Ltd.)
- Two (2) years on Linak actuator
- One (1) year for accessories

Accessories include:

Hand Control

Table Top

Mattress

Safety Belts

Wheels

Defective parts must be returned to Can-Dan Rehattec Ltd. All parts replaced under this warranty become the property of Can-Dan Rehattec Ltd.

The warranty specifically excludes:

- Normal wear and tear items.
- Labour, parts and lubricant costs of all maintenance services.
- Installation done by a non-authorized representative.
- Damage resulting from improper repairs, modification or use of non-approved part, or repairs done by a non-authorized Can-Dan Rehattec representative.
- Damage resulting from accident, fire, theft, vandalism or any Act of God.

Serial No: _____

Signature: _____ Date: _____