



## **INSTRUCTION GUIDE**

**READ THOROUGHLY BEFORE OPERATING**

### **MOBILE ELECTRIC HEIGHT ADJUSTABLE CHANGE TABLE**

- ☐ **Model # 19-97-62**  
**Size: 62" x 28"**
- ☐ **Model # 19-97-72**  
**Size: 72" x 28"**
- ☐ **Model # 20-02-62**  
**Size: 62" x 30"**
- ☐ **Model # 20-02-72**  
**Size: 72" x 30"**
- ☐ **Model # 20-03-62**  
**Size: 62" x 32"**
- ☐ **Model # 20-03-72**  
**Size: 72" x 32"**

**Manufactured in Canada by Can-Dan Rehatec Ltd.**

3-1378 Sandhill Drive, • Ancaster, Ontario • L9G 4V5

---

www.can-dan.com • info@can-dan.com • Tel: (905) 648-7522 • Fax: (905) 648-7799



# **Mobile Electric Height Adjustable Change Table**

## **Instruction Guide**

### **Table of Contents**

1. Safety Precautions
2. Scope of Application
3. Product Information
4. Operation and Handling Instruction
5. Maintenance
6. Trouble Shooting
7. Warranty
8. Change Table Fact Sheet

## **Safety Precautions**

### **Before using, always make certain:**

- Care-giver has received appropriate instruction and training.
- The Instruction Guide has been carefully studied and understood.
- The wheels are in the locked position.
- The user is secured to the table with the 2 integrated safety belts.
- At any time - **DO NOT** store anything under the table-top.

### **NEVER LEAVE USER UNATTENDED ON THE TABLE.**

- When raising the Mobile Change Table, be certain to keep each end of the table sufficiently away from side walls, as the Change Table rises at a slight left to right angle (Total 12").
  - Requires a GFI receptacle (15 amp).

**Maximum Lifting Capacity: 450 Lbs.**

## **Scope of Application**


The Mobile Electric Height Adjustable Changing Table is a lifting device.

Through the use of an electrically powered hand-control, which is connected to a 24 V actuator; the table is lowered to the height of the wheelchair enabling the caregiver to provide transfer assistance onto the table in a safe manner.

Once on the table, the individual can be secured with the two (2) integrated safety belts. The care-giver can then raise the table to the ideal working height.

With a lifting capacity of 450 lbs., it is suitable for use in schools, group homes, private homes, clinics and nursing homes and other care-giving situations.

**Product Information**

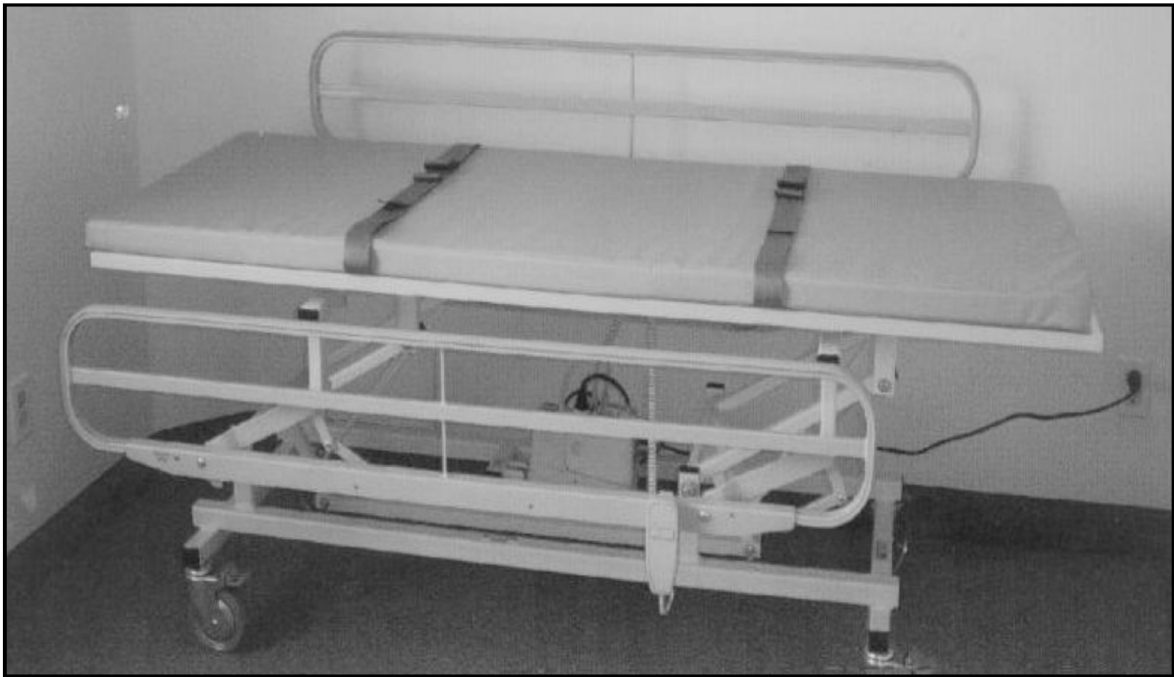

Above picture shows Change Table with optional “Side-Rails”

Model #	Size	Model #	Size	Model	Size
19-97-62	62" x 28"	20-02-62	62" x 30"	20-03-62	62" x 32"
19-97-72	72" x 28"	20-02-72	72" x 30"	20-03-72	72" x 32"

**Specific Features**

- Adjustable in height from 22 inches to 37 inches
  - The padded top comes complete with two (2) body safety belts.
  - Electric hand control fastens to a special hanger on front of the table when not in use.
- Casters with brakes for safety.  
Powered by a 115VAC to 24VDC actuator.  
Mattress is germ resistant, fire retardant and made of vinyl for easy cleaning

## Operating and Handling Instructions

- Using the hand control the care-giver lowers the table to a height where they can easily and safely transfer individual from wheelchair to padded table top.
- Upon completion of transfer, care-giver must secure individual using the 2 (two) provided integrated body safety belts.
- Once individual is safely secured to change table, care-giver can proceed to raise table to a comfortable working position.

### Before using always make certain:

- ☐ Care-giver has received appropriate instruction and training.
- ☐ The Instruction Guide has been carefully studied and understood.
- ☐ Make certain that the wheels on the Mobile Change Table are in a locked position at all times when in use.
- ☐ At any time **DO NOT** store anything under the table top.
- ☐ When raising the Mobile Change Table, be certain to keep each end of the table sufficiently away from side walls, as table raises at a slight left to right angle - total of (12").

**\*Requires a GFI Receptacle (15 amp)**

**NEVER LEAVE USER UNATTENDED ON TABLE**

## Maintenance for Mobile Change Table

The following are to be checked on a regular basis:

- ✓ Safety belts for signs of wear.
- ✓ Wheels and brakes for proper functioning.
- ✓ Clean as needed to ensure proper functioning.

### Yearly Maintenance - Check list

- ☐ Safety Belts
- ☐ Wheels include 2 way brakes
- ☐ Clean velcro on table top and mattress
- ☐ All moving parts to receive a few drops of lubricating oil  
(**Do not use WD40**)

Date: \_\_\_\_\_

## **Trouble - Shooting**

### **If the table fails to operate....**

#### **Check the following:**

- The table is plugged into the AC outlet properly
- There is power to the AC outlet
- Green light located on power supply is lit (indicating power from your AC outlet)

### **If the hand control fails to operate....**

#### **Check the following:**

- Make certain that the hand control is plugged into the power supply box properly.

❖ For other service related problems, contact your local Can-Dan Rehatec Ltd. representative.

## Description of **WARRANTY**

Can-Dan Rehatec Ltd. as manufacturer warrants from the **DATE OF DELIVERY TO THE FIRST CONSUMER** that each item described below, as **NEW AND UNUSED**, will be free from any defects in material and/or workmanship for a period of:

Two (2) years on Change Table Frame.

(Manufactured by Can-Dan Rehatec Ltd.)

1 (one) year on Linak Actuator

Six (6) consecutive months for accessories

Accessories include:	Hand Control
	Table Top
	Mattress
	Safety Belts
	Wheels

Defective parts must be returned to: **Can-Dan Rehatec Ltd.**

All parts replaced under this warranty become the property of:

**Can-Dan Rehatec Ltd.** Ancaster, Ontario, Canada

The warranty specifically excludes:

Normal wear and tear items.

Labour, parts and lubricant costs of all maintenance services.

Installation done by a non-authorized representative.

Damage resulting from improper repairs, modification or use of non- approved parts or, repairs done by a non-authorized **Can-Dan Rehatec** representative.

Damage resulting from accident, fire, theft, vandalism or any Act of God.

Power Supply No: \_\_\_\_\_ Serial No: \_\_\_\_\_

Signature: \_\_\_\_\_ Date: \_\_\_\_\_

<b>Mobile Model</b>	<b>Length: 62"</b>	<b>Length: 72"</b>
<b>Matt Size: Width</b>	<b>28" / 30" / 32"</b>	<b>28" / 30" / 32"</b>
<b>Height Adjustable Range</b>	<b>22" - 37"</b>	<b>22 - 37"</b>
<b>Space Requirement Length</b>	<b>74"</b>	<b>84"</b>
<b>Space Requirement Width</b>	<b>28" / 30" / 32"</b>	<b>28" / 30" / 32"</b>

**NOTE:** All tables require a GFI Receptacle (15 amp)  
 For the Mobile Change Table the outlet should be no higher than 16" from the floor to the top of the outlet and centered to the position of the table.

## **Manufactured by**

**CAN-DAN REHATEC LTD.**

**3-1378 Sandhill Drive**

**Ancaster, ON L9G 4V5**

**Phone: (905) 648-7522**

**Fax: (905) 648-7799**

## **Dealer**

--