

Product Item number	Shower and changing table 2000, mesh cover, electrically height adjustable R8478000-93		
Color Product description	EAN: 5708590318530 FDA: INQ/D124319 White Shower and changing table 2000, powered, height adjustable 27.6"; from 11.8" - 39.4", with wired hand control Foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 35.4" x 70.9". Excl. mounting kit. See Technical specifications (pdf) for information on mounting kits suitable for various wall types. <i>Replaces R8468.</i>		
Adjustability	Height adjustable		
Adjustment height	700 mm		
Product capacity	Maximum load: 385 lbs. When the table is loaded with 385 lbs, each mounting bolt supports 51 lbs. This product has been tested statically with a load of 970 lbs as the IEC 60601- standard requires testing with a safety factor.		
Speed	Speed up: 15 mm/s Speed down: 20 mm/s		
Noise level	48 dB		
Materials	Powder-laquered stainless steel, mesh cover (PVC-coated for extra durability) and polystyrene.		



Product Item number	Shower and changing table 2000, mesh cover, electrically height adjustable R8478000-93		
Ergonomy	Pressalit offers a range of high quality, fixed or power adjusted shower and changing tables designed to provide a stable platform suitable for changing adults and children in a variety of settings: schools, day care centers, institutions, hospitals and private homes. The 2000 series changing table has a low start position to make transfers to the table easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath. The flexibility of the changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.		
Features	A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space in front of the unit. Electric with wired hand control – the red buttons clearly indicate "Up" and "Down". The wired hand control comes with a holder for fitting to the wall. The stretcher of the table is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement.		
Mains connection (V)	120		
Actuator (motor)	LA31 Careline, (Linak), 6000 N, IPX6		
Control box	CB9 Careline, (Linak), IPX6		
Wired remote control	HB3 (Linak), IP66		
Power consumption (maximum load)	Power consumption by maximum load: 70 W Power consumption by standby: 2 W		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.		



PRESSALIT.

Product Item number	Shower and changing table 2000, mesh cover, electrically height adjustable R8478000-93
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: CaviCide The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.



Product Item number Shower and changing table 2000, mesh cover, electrically height adjustable R8478000-93

Mounting in different wall types

0		
	Concrete >B25 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Solid Brick >= Mz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Solid Sand Lime Brick >= HIz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.	
	Aerated Concrete P4 V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.	



Product Item number	Shower and changing table 2000, mesh cover, electrically h R8478000-93	eight adjustable
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.



www.pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

