

Meywalk® 2000		Item #	Website Category	
Manufacturer	Meyland-Smith			
Description	The Meywalk® 2000's stands out in the market of assistive technology by being spring suspended. The springs stimulate the muscles of the user and the movements that are a part of a natural gait are imitated. This way the gait pattern is developed. The seat unit can easily be lowered, making entry and exit easier for the user. The body posture of the user can be set in any position from vertical to 15° forward tilt, making it possible to find the body inclination that gives the user the best control. The Meywalk® 2000 is made with materials of high quality, which guarantee long life, maximum reliability and minimum maintenance.			
Features	<ul style="list-style-type: none"> • Easy adjustment • Dynamic spring action • Promotes natural gait • Wide range of accessories • Adjustable angle handlebar encourages more upright or forward trunk orientation • For Indoor and Outdoor Use (on hard, flat surfaces) 			
Specs		Small	Medium	Large
	Length (cm)	85	95	95
	Width (cm)	68	70	70
	Seat/groin height (cm)	48 - 66	62 - 86	76 - 100
	Trunk support height (cm)	81 - 100	94 - 117	108 - 131
	Wheel size, front (mm)	150 x 50	200 x 50	200 x 50
	Wheel size, rear (mm)	400 x 50	400 x 50	400 x 50
	Weight (kg)	22	25	26
	Maximum user weight (kg)	100	100	100
Youtube URL				
Product PDF Sell Sheet				
Product Image	<p style="text-align: center;">Ease in Transfers!!!</p> <p>To lower the seat and allow for transfer, first lock the unit, lower the tail gate and open the ring. Press the two silver buttons on the front of the walker and raise the lift bar which lowers the sit bar. After the user transfers, close the ring and tail gate, and push down on the lift bar. The lever lift bar easily lifts the user into standing!</p>			