



“FOLD UP”

Wall Mount Electric Height Adjustable Change Table

Model #20-00-62
Size: 62” x 24”

Manufactured by

**CAN-DAN REHATEC LTD.
7-1469 Sandhill Drive
Ancaster, ON L9G 0H7
Phone: (905) 648-7522
Fax: (905) 648-7799
Email: info@can-dan.com**

Dealer



INSTRUCTION GUIDE

V 1.2

Read Thoroughly before Operating

Manufactured in Canada by Can-Dan Rehatec Ltd.



Description of WARRANTY

Can-Dan Rehatec Ltd. As manufacturer warrants from the **DATE OF DELIVERY TO THE FIRST CONSUMER** that each item described below, as **NEW AND UNUSED**, will be free from any defects in material and/or workmanship for a period of:

- Four (4) years on Change Table Frame
(Manufactured by Can-Dan Rehatec Ltd.)
- Two (2) years on Linak Actuator
- One (1) year for accessories

Accessories include: Hand Control
 Table Top
 Mattress
 Safety Belts
 Front Cover

Defective parts must be returned to **Can-Dan Rehatec Ltd.**
All parts replaced under this warranty become the property of **Can-Dan Rehatec Ltd.**

The warranty specifically excludes:

- Normal wear and tear items;
- Labour, parts and lubricant costs of all maintenance services;
- Installation done by a non-authorized representative;
- Damage resulting from improper repairs, modification or use of non-approved part or repairs done by a non-authorized Can-Dan Rehatec representative;
- Damage resulting from accident, fire, theft, vandalism or any Act of God.

Serial No: _____

Signature: _____ Date: _____

Maintenance

Check the following regularly:

- Check safety belts for signs of wear
- Clean Velcro on table top and mattress

How to remove front acrylic cover:

- Remove table top
- Unscrew two ¼" bolts on top of front cover
- Unplug the hand control from the power supply box

Replacement of Hand Control:

- Follow instructions above on how to remove front cover
- Remove two (2) screws on the side of the front cover (top and bottom)

Lubrication of Wall Mounted Frame

One (1) time each year, REMOVE the front cover (follow instructions above) and grease the steel track. Only use "Super Lube" Synthetic Grease with Syncolon.

Check list for yearly maintenance

Date performed: _____

- Safety belts
- Clean Velcro on table top and mattress
- Lubricate track (follow instructions above)

*** Copy this sheet and keep for your file**

TABLE OF CONTENTS

1. Safety Precautions
2. Scope of Application
3. Lock Box & Safety Lock
4. Safety Lock
5. Installation Instructions
6. Operation and Handling Instructions
7. Maintenance
8. Warranty

Safety Precautions

IMPORTANT

Please read these instructions carefully, note the safety instructions and warnings. Use the product correctly and with care for the purpose for which it is intended. Failure to do so may cause damage to property and/or personal injury. Please keep this instruction manual safe for future use.

Before using, always make certain:

- ✓ Caregiver has received appropriate instructions and training
- ✓ The Instruction Guide has been carefully studied and understood

SAFETY INSTRUCTIONS AND WARNINGS:

- ✓ Use a qualified person to maintain the table in good condition. Keep it clean for best and safest performance.
- ✓ Use this table for lifting purposes only. **DO NOT** use it for any other purposes it is not designed to perform.
- ✓ Keep children and unauthorized persons away from the work area.
- ✓ **DO NOT** make any modifications to the lift
- ✓ Ensure the individual is secured to the table using the two (2) integrated safety belts provided
- ✓ **DO NOT**, at any time, leave anything under the change table

Requires a GFI receptacle (15 amp)

Maximum Lifting Capacity: 250 lbs

NEVER LEAVE USER UNATTENDED ON TABLE

WARNING: The warnings, cautions and instructions discussed in this instruction manual cannot cover all possible conditions and situations that may occur. It must be understood by the operator that common sense and caution are factors which cannot be built into this product, but must be supplied by the operator.

Operating and Handling Instructions



- Unlock safety lock bracket (support bracket with padlock)
- Unfold table (holding onto table at all times)
- Using the hand control, the caregiver lowers the table to a height where they can easily and safely transfer the individual from wheelchair to padded table top.
- Upon completion of transfer, the caregiver must secure the individual using the two (2) integrated body safety belts provided.
- Once the individual is safely secured to the change table, the caregiver can proceed to raise the table to a comfortable working position.
- When finished using the change table, FOLD the table into it's upright position. Use the safety lock bracket to hold and secure the table top, lock with supplied padlock.

Before using, always make certain:

- The Instruction Guide has been carefully studied and understood.
- The caregiver has received the appropriate instruction and training.
- The individual is secured to the table with the two (2) integrated safety belts.
- At any time **DO NOT** store anything under the table top.

Requires a GFI receptacle (15 amp)

NEVER LEAVE USER UNATTENDED ON TABLE

Installation Instructions

For Load-Bearing Block Wall

- Place MAIN frame against the wall
- Adjust the legs to get the frame in level (use a level)
- Hold the frame against the wall and pre-drill with 3/8" drill and install "Single Expansion Shields"
- Bring the frame back and fasten with 3/8" bolts

Must be installed on a load bearing block wall or re-enforced wall approved by a structural engineer. Engineer/Client of record to validate wall is a structural wall capable of supporting change table with 250lbs weight rating.

Power Supply

Place the front cover up close to the frame and connect the plug from the front cover to the power supply box.

Note: In slot on box

Front Cover

Place the front cover over MAIN frame and fasten with 1/4" bolt.

Scope of Application

The Wall Mount "Fold Up" Change Table is a lifting device. Through the use of an electrically powered hand control, which is connected to a 24V actuator, the table is lowered to the height of the wheelchair, enabling the caregiver to provide transfer assistance onto the table in a safe manner.

Once on the table, the individual must be secured using the two (2) integrated safety belts. The caregiver can then proceed to raise the table to their ideal working height.

With a lifting capacity of 250 lbs., it is suitable for use in schools, clinics, private homes, and other caregiving situations for children.

Lock Box & Safety Locking Bracket



Close-up of Safety Locking Bracket

